

Power-Vac



Thousands of Hearing aid wearers clean and maintain their hearing aids with Power-Vac.

Developed after observing hundreds of hearing aids being sent for repair for clogged receiver or microphone ports. Power-Vac will pay for itself, the cost is much less than repairing and shipping a hearing aid one time. Thousands of people use Power-Vac to maintain all styles of hearing aids including Open Fit models.

Easy steps for cleaning a hearing aid

- a. The entire hearing aid should be brushed to remove any loose wax or debris. The wax-loop on the Combination Cleaning Tool can loosen or remove wax from the receiver port. The vent should be cleared with the vent cleaning tool.
- b. Use the cone shaped suction tips to clear receiver and microphone ports by matching the diameter of the tubing with the cone shaped suction tip.
- c. The hearing aid (or earmold) is then placed on the standard suction tube to further remove loose wax and debris from the hearing aid with a combination of suction and vibration. The standard tube can also be used to “vacuum” the entire surface of the hearing aid.
- d. Brush the hearing aid and wipe it with a dry tissue or paper towel prior to use.